



# Instructions for Use


## Initial Impressions of Denture(s)

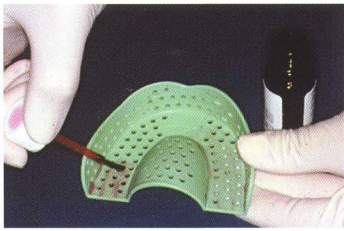

Paint trays with adhesive

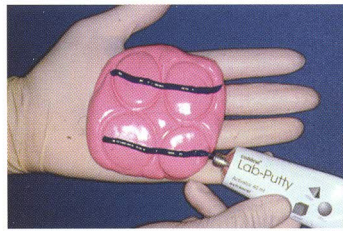


Measure putty and activator  
4 Scoops + 12cm  
(1 Scoop diam = 3cm)

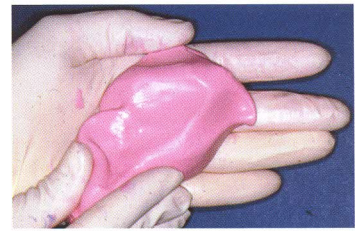


Mix putty until catalyst is  
evenly distributed



Press putty into tray, smooth  
and embed denture

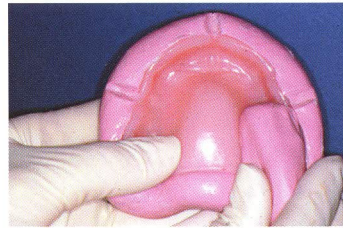


When set, cut grooves and  
smear with hand cream

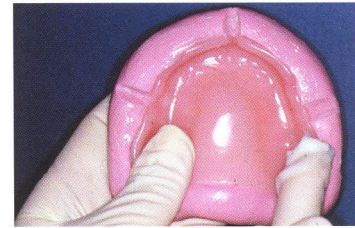

Mix putty and complete  
mould as Fig 2

## Working Impressions

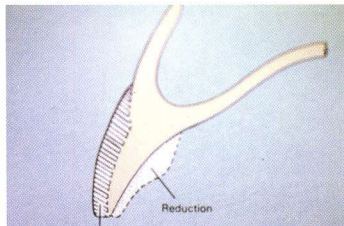

Make any necessary  
alterations to occlusal, incisal,  
labial or buccal surfaces




Dry Replablocks and paint shellac  
and labial borders of Replablock  
with adhesive. Air dry excess.



Measure Coltène Rapid Liner  
(5 or 10ml upper, 5ml lower)



Measure Activator  
by volume.  
Trickle down mixing rod

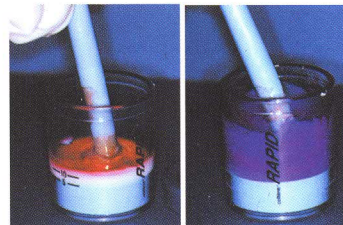


Mix well until material  
changes to deep  
purple colour



Clip lid on mixing pot and  
dispense onto fit surface  
of Replablock

## Record of Jaw Relationship

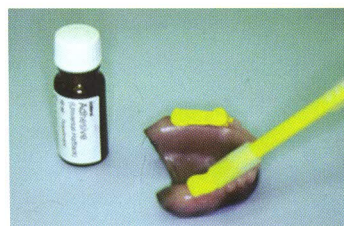

Paint adhesive on one occlusal  
surface. Apply President Jet Bite or  
similar, insert and record.